

# Using colostrometer




**Prepare cows udder for milking**



**Collect the first milking colostrum in a clean dry container**



**Fill the measuring cylinder 2/3 full (1/2 ltr) with colostrum**

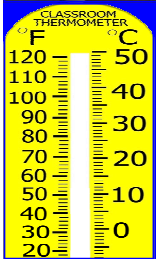


**22°**

**Allow the colostrum in the cylinder to cool to room temperature (22° C)**



**Gently place the measuring device into the cylinder filled with colostrum. The measuring device should float freely in the cylinder.**

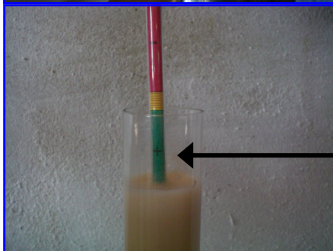

**Determine the colostrum quality using the color coded scale inside the colostrum measuring device.**

**High quality**

**◀ Feed colostrum for first feeding, or store for future use**

**Medium quality**

**◀ Feed or store colostrum for second feeding**

**Poor quality**

**◀ Feed colostrum to calves 1 day or older. Do not store !**